DE

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EN

Cleaning and care:

- All cleaning agents suitable for smooth surfaces can be used for cleaning.
- The casters must occasionally be cleaned to ensure the function.
- **Important:** When using disinfectants, the manufacturer's instructions for use must be observed.

⚠️ Safety instructions:

- Observe the maximum permissible payload!
- Do not hang clothes or other objects, that are not intended to be used for the care of the patient, on the IV Pole.
- You must not stand on the cross with one or both feet!
- The rollers are only suitable for smooth and even surfaces.
- The IV Pole is stable to an inclination angle up to 6 degrees (under max. load). An inclination over 6° or partial loading with an unfavorable load distribution causes a risk of tipping over.
- During transport, the IV Pole must always be guided to the pull-out tube by hand with one hand at the extension tube.

Warranty:

According to the legal warranty claims. From the warranty excluded are damage by inappropriate use, normal wear or use as well as defects, that affect the value or usability of the device only insignificantly. Technical changes, alterations and extensions and improper maintenance work lead to expiring the warranty and the product liability. We reserve ourselves changes, which serve the technical improvement.

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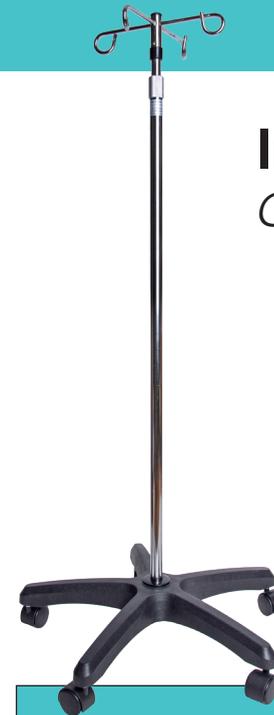
 Valentine International Ltd., 8th Floor, No. 149, Sec 2, Ta Tung Road,
221 Hsichih District, New Taipei City, Taiwan R.O.C.

 Y. Sung Handelsvertretung, Toulouser Allee 9, DE-40211 Duesseldorf, E-Mail: y.sung@t-online.de

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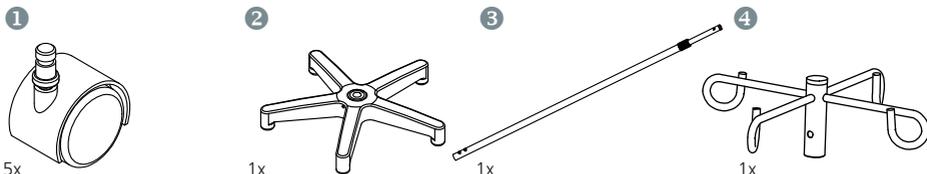
EN

Intended purpose:

The infusion stand is a medical device and is used to hang infusion bags, bottles, containers, etc. for infusion. The (brake) rollers allow mobility and parking possibilities.

Indication:

For use in enteral diet and parenteral infusion/medicine

Contraindication:

Insufficient mental abilities to use

Before use / maintenance:

Please read this instruction manual in full first and keep it until the end of the product life. Check the delivered parts for completeness, before first use and any use should be checked to make sure that the infusion stand is undamaged and functional. Defective or improperly installed products may not be used. Any use other than or beyond the above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents arising in connection with this product must be reported to the manufacturer or the competent authority without delay.

Assembly:

The IV Pole is delivered unassembled. The assembly is done without tools by simply connecting the individual parts:

1. Stick the 5 casters ① in the provided holes at the bottom of the base ②. In doing so you have to pay attention that the both braking castors assembled opposite, not side by side (brake efficiency).
2. Now the stand tube ③ will be stuck in the provided holes at the upper side of the base. The push button at the tube gives stability through snapping in the provided hole.
3. Finally you stick the bottle holder ④ at the top of the stand tube.
Again the push button gives stability by snapping in the provided hole.

Disassembly:

The infusion pole can be disassembled according to the assembly in reverse order.

Pumps and utensils for controlling the infusion flow

are allowed to be installed in the lower third only (please pay attention to the max. payload capacity!).

Height adjustment:

- Hold the extension tube
- 4. By turning the screwing devices ⑤ to the left, the connection of the both inserted tubes loosen and are able to get the desired height through pulling or pushing.
- Afterwards screw tight the screwing device by turning right.
- **Caution:** If the pull-out pipe is not held down or released before the tightening screw is tightened, it slides suddenly downwards and can lead to damage.

Braking rollers:

Always use **both** lockable casters by pressing them down. To release the brakes, push both brakes up again.

Mobility:

The IV Stand can be carried along without problems on smooth, level surfaces. For this, both brakes must be loosened first! In case of inclination of up to 6°, the infusion stand is stable. Over 6° the extension tube must be retracted completely. During transport, the IV Pole must be guided with one hand. Do not place the IV Pole on an inclined plane. (See also safety instructions).