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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Murano toilet chair is a medical device. It is a technical aid for personal hygiene intended for elderly people, those with reduced mobility, disabled individuals, those who are injured, or those who are prone to losing their balance easily. It provides comfortable positioning and WC use for strong individuals. Reference 811025 is height-adjustable.

PRODUCT DESCRIPTION

811024/811025 :

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum weight limit of 130 kg.
- Ensure all components are secure before use.
- Make sure the product is in good working condition.
- Visually inspect the product.
- Check the stability of the chair before use.
- Use only on a flat surface.
- Before each use, ensure that all four legs are at the same height and that the clips are properly engaged (for model 811025).
- Never lean while seated on the chair. The risk of tipping increases with unilateral heavy loading.
- Do not climb onto the toilet chair - risk of slipping!
- Be cautious of pinch hazards during use.
- If any parts are damaged, broken, worn, or missing, contact the dealer.
- Report any serious incidents related to the product to the manufacturer and the relevant authority in the member state where the user and/or patient is located.

MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

INSTALLATION :

- Before use, ensure the commode chair is stable on a flat, horizontal surface.
- To adjust the height of the commode chair (model 811025), remove the clips and slide the leg extension to the desired height. Make sure all legs are adjusted to the same height and reattach the clips.
- Insert the commode bucket. Ensure that the bucket's handle rests on the front bar of the commode chair. Verify that the bucket is properly positioned underneath while the chair is in the seated position and throughout its use.
- Attach the toilet seat to the rear crossbar, ensuring that the seat clicks into place.

USE :

- When moving the product, always lift it by placing your hands on the tubular metal frame under the seat.
- Do not allow children or unauthorized individuals to play with or use the equipment without proper supervision.
- Do not rock back and forth or side to side while using the product, as this could cause excessive stress and potentially lead to damage or failure, which may result in injury.
- Do not use the product outside the home or on uneven surfaces (such as in the garden), as this could put stress on the legs and potentially cause damage.
- Do not insert fingers or foreign objects into exposed tubular sections, as this may cause serious injury.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with a damp cloth and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth.
- Before cleaning the chair, remove the two armrests, the backrest and the four extension feet with the end caps. Clean these parts separately from the main frame.
- The chair must be completely serviced every year, with compulsory lubrication.
- It is recommended that the product be serviced at least once a year. If the product is subject to intensive use and/or use in a multi-user environment, we recommend that it be serviced more frequently.
- Before each use, check the condition of the rubber feet, the frame, and the seat.
- In the event of a malfunction, do not repair the product yourself. Consult an authorized dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

811024 :

Maximum load capacity (kg)	130
Weight (kg)	8
Total width (cm)	60
Height (cm)	87
Depth (cm)	55
Seat width (cm)	43
Seat depth (cm)	45
Seat height (cm)	51
Color	Blue
Seat opening size (cm)	24 x 21
Frame	Powder-coated steel
Seat	Wood + Foam + PVC
Feet (or tips)	TPE
Bucket	PP

811025 :

Maximum load capacity (kg)	130
Weight (kg)	8
Total width (cm)	59.5-62
Height (cm)	82-98
Depth (cm)	53-61
Seat width (cm)	43
Seat depth (cm)	45
Seat height (cm)	46-62
Color	Blue
Seat opening size (cm)	24 x 21
Frame	Powder-coated steel
Seat	Wood + Foam + PVC
Feet (or tips)	TPE
Bucket	PP

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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