BARRIERE DE LIT ESCAMOTABLE

ADJUSTABLE BEDRAIL

REF 823002

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
INSTRUCTIONS MANUAL EN

Warning:

1. Maximum user weight: 150kg weight limit
Do not exceed the weight limit stated as doing so could put the user at risk
2. Please examine fully for any damage or faults and read the manual carefully before using the bed rail. If you suspect a fault or damage, do not use and contact your supplier immediately.
3. Do not use it without any mattress or the thickness of mattress less than 10CM.
4. Clean regularly with soft cloth and do not use abrasive cleaners.


Assemble instructions:

Step 1: This bedrail can be placed on both sides of the bed. In this figure, the bed rail is showed on the left side of the bed.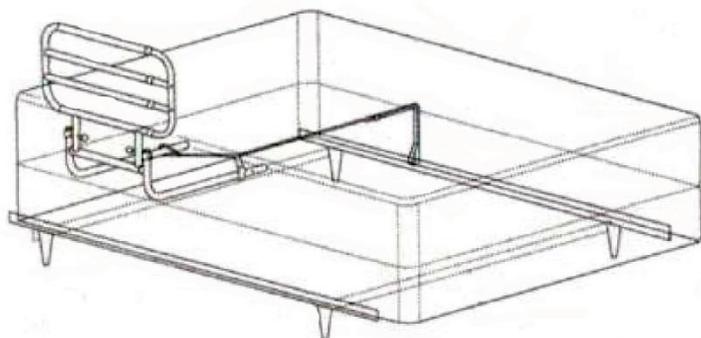

Step 2: Remove all parts from the box and place them on a flat surface.

Step 3: Carefully check that all parts are included in the carton.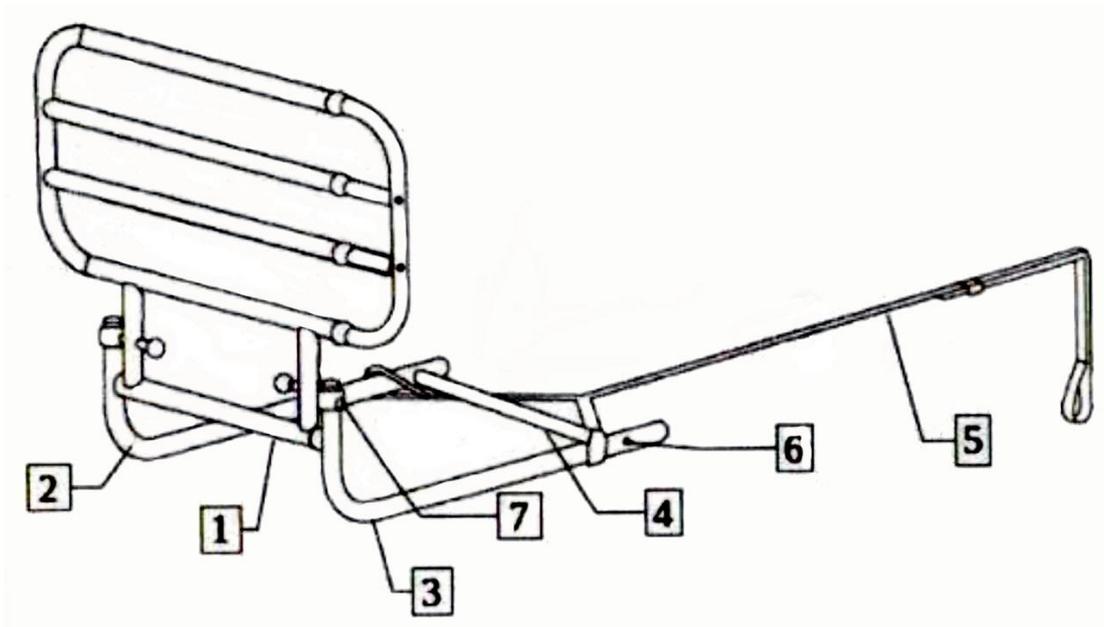

- 1) Main structure
- 2) L shaped tube
- 3) L shaped tube
- 4) Transversal bar
- 5) Security bar
- 6) Screws
- 7) Washers

Step 4: Secure the L shaped tubes (2&3) with screws and washers (6&7) to the main structure (1)

Step 5: After that, place the security belt (5) in the L shaped tubes (2&3), as shown in the picture. Make sure the tapes of the security belt are in front of the transversal bar (4) so that they won't slip.

Step 6: Finally fixed the transversal bar (4) with the screws (6) and washers (7) to each bar

Technical Specifications

Material	:	Steel
Adjustable length	:	67cm to 107cm
Height	:	58cm