TABLE DE LIT SIMPLEX LIFT

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a walking aid. He/she will advise and inform you on the most appropriate walking technique for you.

Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

The Simplex Lift bed table is a medical device. This product lets you read and eat in bed in complete comfort. The height and tilt of the table top can be adjusted to suit different times of the day. The height can be easily adjusted by means of a compressed air cylinder, which locks into place in stages.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to observe the precautions for use may result in the risk of serious injury.


- Use the bed table only as described in this manual.
- Do not exceed the maximum weight of 15 kg
- Install the bed table on a flat surface.
- Store the bed table indoors, away from mould and dust.
- Check that the adjustment knobs work properly before use.
- Before use, check that all parts are present and in good condition. In the event of damage, please contact your dealer.
- Carefully follow the assembly instructions and ensure that all components are correctly secured before use.
- Before use, apply the brakes to the castors.
- If any parts are damaged, broken or worn, please contact your dealer.

MOUNTING AND USE


ASSEMBLY AND USE :

Remove all the components from the carton and check that each part is present and in good condition.


1. Assemble the wheels on the base (Figure A). Wheels with brakes must be fitted at the rear of the table.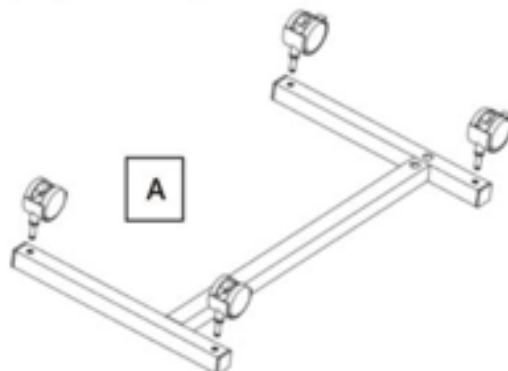

2. Install the cylinder: lay the tray flat, with the side with all the threads facing upwards. Position the cylinder as shown in Figure B. Prepare the screws to be fitted to the underside of the plate, by first fitting the split washer and then the single washer. Screw in each screw and do not tighten them until all have been fitted.

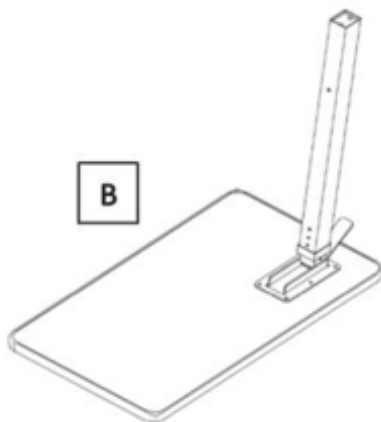
3. Install the base: align the holes in the base with those in the cylinder. Position the assembly plate and secure the assembly using the screws on the base/cylinder. Note that the foot must be installed in the same direction as the platter (Figure C).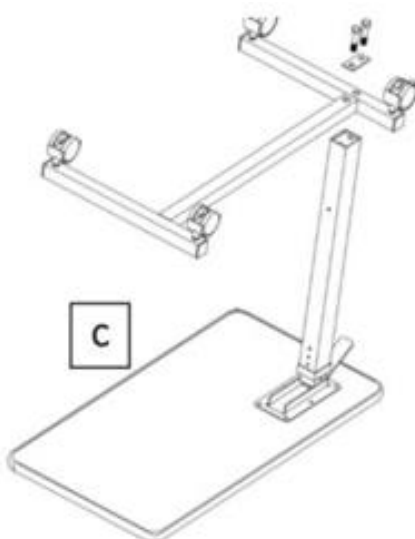


Adjusting the table :

1. When using the table for the first time, remove the black screw on the cylinder. Do not remove the grey screw. Keep the black screw in a safe place, as it may be useful for dismantling in the future.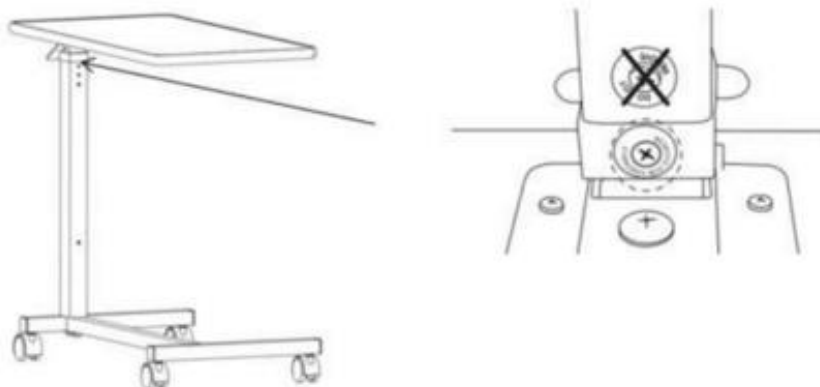

2. To adjust the height of the table, press the lever on the cylinder and then raise or lower the table top to the required height.

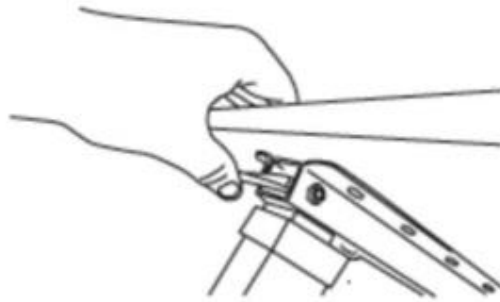CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water, a soft cloth and a neutral detergent.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Dry with a soft, clean, dry cloth.
- Make sure there is no accumulation of dirt on the product.
- Periodically check all screws for wear and strength. Replace all broken, damaged or worn parts immediately.
- Regularly check the operation of the brakes on the castors and the operation of the jack.
- In the event of a malfunction, do not repair the table yourself. Consult an authorised dealer.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place too heavy objects on the product components, do not jam anything...).

TECHNICAL SPECIFICATIONS

Dimensions	Tray dimensions: width 76.3 x depth 38.3 x height 72 to 99.5 cm. Leg dimensions: width 66 x depth 38.8 cm. Clearance width: 61 cm.
Weight	9 kg
Maximum weight supported	15 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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