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1 - INDICATIONS

- Indicated for inguinal hernias (designed for the use of extra pads #OT6)* and post-operative treatment
- Indicated post-operatively to prevent the sagging of abdominal tissues
- Indicated as a support during sports and heavy work activities
- Choose carefully the model with the most appropriate height and elastic force for your case to achieve the best results

2 - CONTRA-INDICATIONS AND SIDE-EFFECTS

- A medical advice is necessary before using the medical device
- Do no use the hernia brief in case of strangulated and/or incarcerated hernia, inflamed or non-reducible hernias
- Be careful to exclude genitals from the contention area
- Since this device is NON-INVASIVE and made with non-allergenic fabrics, no negative or side effects have ever been noted.
- Any serious incident occurred in relation to the device use, should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

3 - MAINTENANCE AND CARE

This device is non-biodegradable. Even though it is made of natural fabrics, after use, avoid release the product to the environment.
AT THE END OF ITS USABLE LIFE HAS TO BE DISCHARGED ALONG WITH SOLID ORDINARY WASTE.

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