 **APPUI DE LIT LATERAL ROBUST
– M210**

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 **ROBUST LATERAL BED SUPPORT
- M210**

Instructions for use

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IMPORTANT

Read these instructions before use. Keep the packaging and these instructions for future use. The locking pins on the feet must always be fully engaged in the holes in the tubes.

MAXIMUM LOAD = 65 kg

This device is designed to provide support and stability for the user. It is under no circumstances intended to support the total weight of a person.

NOT FOR USE IN NURSERIES AND PAEDIATRICS

SAFETY PRECAUTIONS

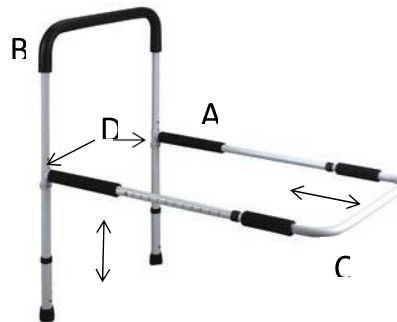
Before getting started

- Read the installation and operating instructions.
- Check the contents of the package
- Make sure you have the right equipment and sufficient clean space to carry out the installation.

PRODUCT DESCRIPTION

List of components :

- A. Bed rail x 1
- B. Support handle x 1
- C. Bed rail support x 1
- D. Collar pins x 2

ASSEMBLY & USE

ASSEMBLY :

- Assemble bed rail A with support handle B. Secure the assembly using the collar pins D and make sure that the collars click perfectly onto the metal tube.
- Assemble the structure in 1 with the crossbar support C. To do this, push the two parts together, pressing the locking pins simultaneously. Adjust the assembly according to the width of the mattress (ideally 2/3 of the width of the mattress) and lock by sliding the tubes so that the fingers fit perfectly into the holes in the tubes.
- Remove the mattress from the bed (get help if you need it).
- Place the device on the base and position it along the bed in the desired position.
- Replace the mattress on top of the installed device (ask for help if necessary).
- Adjust the height of the support handle by removing the collar pins D and sliding up or down to the required height. Replace the pins in the holes and make sure that the collars fit perfectly on the metal tube (the end of the pin opposite the collar must come out of the hole in the tube).
- Adjust the height of the legs by pressing down simultaneously on the locking fingers and sliding the legs to adjust the assembly to the correct height. Make sure that the locking fingers fit perfectly into one of the holes in the tube. The 2 legs must be adjusted to the same height.

SERVICING - MAINTENANCE

Clean regularly with a standard cleaning detergent. Never use abrasive products. Periodically check the assembly, especially the fasteners.

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